Instructions for use Silicone Breast Milk Collector

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Read all instructions before use SAVE THE INSTRUCTIONS FOR USE

Intended Purpose/ Intended User

When the mother is breastfeeding or pumping on one side, the Medela Silicone Breast Milk Collector can help to collect the leaking breast milk on the other side.

Important safety information/ Residual Risk/Warnings

- When using the silicone leak-proof stopper, make sure that it is inserted tightly in order to prevent milk from leaking.
- During use, make sure that the Silicone Breast Milk Collector doesn't press on the milk ducts of the nipple. First ensure the nipple is centered into the shield when applying to the breast.
- The Silicone Breast Milk Collector may lose suction and separate from the breast on occasion due to capacity, skin humidity, etc.
- Keep the product and accessories away from babies and children.
- The product and its packaging are not toys, so do not allow babies and children to play with them.
- The lanyard is solely designed and intended to be worn by adults. Do not place it around baby's / children's necks.
- Never leave a baby or child unattended with the lanyard.
- Do not let the baby chew on the product or its accessories.

How to Use

- 1 Whenever using the Silicone Breast Milk Collector ensure the lanyard is attached and worn around the neck to prevent accidental spills during use.
- 2 Squeeze in the center of the collection bottle. Firmly place the shield against your breast and areola, with your nipple centered to form an airtight seal. You can then start breastfeeding or pumping.
- 3 Once the Silicone Breast Milk Collector is full or when you have stopped pumping / feeding, hold on to the collector and gently squeeze the shield to remove it from your breast.
- 4 To ensure breast milk doesn't spill insert the stopper before placing it down.
- 5 When you are finished, place the collector on the table and push down a little to create suction on the base, which will firmly stick it to the table surface, making sure it does not fall over.
- 6 When you want to move the collector, break the seal on the table surface by gently lifting up the tab on the suction base while holding onto the collector with the other hand.
- 7 When you need to store breast milk for a short time, place the lid on top, and store it at room temperature or in the refrigerator. For information on storing your breast milk, visit BreastMilkGuidelines.com (U.S.) or Medela.ca/Breastfeeding/Moms-Journey/Storing-and-Thawing-Breast-Milk (Canada)

Cleaning Instructions at Home

- Remove the lanyard from the collector before cleaning. Do not clean with harsh chemicals.
- Before the first use, thoroughly rinse and clean the collector.
- Place all parts except lanyard into a pot, and fill with cold water (distilled water is recommended) until all parts are fully submerged.
- Bring the water to a boil and allow it to continue boiling for approximately three minutes.
- Place parts on a clean surface and/or towel and allow parts to air dry. Do not store wet or damp parts.

After each use:

- Always wash all parts as soon as possible after each use. Parts can be washed with a mild dish soap and then rinsed thoroughly in cold water. Do not use a baby bottle brush.
- Check the Silicone Breast Milk Collector before each use. Do not use if there is any noticeable wear or damage.
- The collector is made of silicone, which is a material that is easily oxidized, so the material may turn yellow after using for a long time. This will not impact the function of the parts.
- Make sure the products is not placed near any sharp objects when storing.

Disinfection

- Before use, clean the parts according to the cleaning instructions.
- The Silicone Breast Milk Collector is dishwasher safe on the top rack only.
- It is safe to sanitize in the Medela Quick Clean™ Micro-Steam™ Bags.

Disposal

According to local regulations.

Storage/Transportation/ Environmental Conditions

When you need to store it for a longer period, pour your breast milk into a Medela storage bag or bottle.

Material

Collector and stopper: Silicone;
Cover: Polypropylene; Lanyard: ABS

Meaning of symbols/Acronyms/ Abbreviations

- Indicates the manufacturer
- REF** Indicates the manufacturer's product number
- LOT** Indicates the manufacturer's batch code or lot
- Fragile
- Keep away from sunlight
- Keep dry
- Consult the instructions for use
- pcs** Indicates the number of products included in the package
- Indicates the date of manufacture
- Indicates that the package is capable of being recycled

Medela (Changzhou)
Medical Technology Co., Ltd.
Sino-Swiss International
Industrial Park (Changzhou)
Building A1 No. 2 Chuangye
West Road, New District,
Changzhou 213034, Jiangsu,
China
Customer Service Phone:
400-670-3600
www.medela.cn

Distributed by:
Medela LLC
1101 Corporate Drive
McHenry, IL 60050, USA
Phone (800) 435-8316
or (815) 363-1166
customerservice@medela.com
www.medela.us

Distributed by:
Medela Canada Inc.
4160 Sladeview Crescent, # 8
Mississauga, ON, L5L 0A1
Canada
Phone (905) 608-7272
info@medela.ca
www.medela.ca

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